HydroClean[®] advance

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- [1] Knestele, M (2004) The treatment of problematic wounds with HydroClean advance - tried and tested over many years in clinical practice. Údaje HARTMANN ve spisu.
- [2] Eming S, Smola H, Hartmann B, et al (2008). The inhibition of matrix metalloproteinase activity in chronic wounds by a polyacrylate superabsorber. Biomaterials 29: 2, 2932-2940
- [3] Zöllner, P (2010). Observational study Ideal wound environment during granulation and epithelization achieved with HydroTac. Údaje HARTMANN ve spisu.
- [4] Humbert P., et al. on behalf of the CLEANSITE study group. Protease-modulating polyacrylate-based hydrogel stimulates wound bed preparation in venous leg ulcers a randomized controlled trial. Journal of the European Academy of Dermatology and Venerology 2014; 28:12, 1742-50.
- [5] Spruce P, Bullough L, Johnson S et al (2016) Benefits of HydroClean advance in wound bed preparation: a case study series. Wounds International (v tisku)
- [6] Meuleneire F (2011b) Clinical evaluation of TenderWet plus. Dokument HARTMANN [Údaje ve spisu]
- [7] Meuleneire F (2013) TenderWet plus in treatment of patients with problem wounds- HARTMANN document. Údaje ve spisu.
- [8] Zollinger C, Schwab R, Locher E et al (2014) HydroTherapy. Application study. Dokument HARTMANN. Údaje ve spisu.
- [9] Ousey, K. et al. The importance of hydration in wound healing: reinvigorating the clinical perspective Journal of Wound Care Vol 25, No 3, March 2016.
- [10] Bruggisser, R. (2005). Bacterial and fungal absorption properties of a hydrogel dressing with a superabsorbent polymer core. J Wound Care 14, 438 -42
- [11] Smola H.: Stimulation of epithelial migration - novel material based approaches. Předneseno na kongresu EWMA, 2015, Londýn. Údaje ve spisu: studie in-vivo, H. Smola.
- [12] Courderot-Masuyer, C. et al. (2005) Study on the behavior of healthy fibroblasts and venous ulcers after infection with Pseudomonas aeruginosa and in the presence of a HydroClean active dressing. Journal des Plaies et Cicatrisations. 51: 3-7.
- [13] ISBN Monograph. 978-1-944788-63-6-ISBN Services
- [14] Bullogh, L. Johnson, S. O'Brien, D. (2016) A Multi-Centre 15 patient evaluation of a Hydro-Responsive Wound Dressing (HRWD) - HydroClean advance. Poster prezentovaný na sympoziu o HydroTerapii: A New Perspective on Wound Cleansing, Debridement and Healing
- [15] Kaspar, D (2011). Therapeutic effectiveness, compatibility and handling in the daily routine of hospitals or physicians's practices. Údaje HARTMANN ve spisu: Hydro-Responsive Wound Dressing (HRWD) a AquaClear Technology jsou obchodní známky společnosti HARTMANN
- [16] Ousey, K. et al. Wound Healing and Hyper-Hydration - A Counter Intuitive Model. Journal of Wound Care 2016;25(2): v tisku
- [17] Ellermann, J (2015). HydroClean 2.0: Design validation customer/user interviews. Internal Report, International Marketing Department
- [18] Dowsett C, Claxton K. Reviewing the evidence for wound bed preparation. J Wound Care 2006; 15: 439-42
- [19] Young T. Debridement — is it time to revisit clinical practice? Br J Nurs 2011; 20(suppl): 24-28
- [20] Young T. Reviewing best practice in wound debridement. Prac Nurs 2011; 22: 488-92
- [21] Wound bed preparation: TIME in practice. Wounds UK Caroline Dowsett, Heather Newton, Wounds UK 01/2005; 1
- [22] O' Brian, D and Clarke, Z. (2016) The patient experience with a Hydro-Responsive-Wound Dressing (HRWD) – HydroClean® advance. Poster prezentovaný na sympoziu o HydroTerapii: A New Perspective on Wound Cleansing, Debridement and Healing.
- [23] No additional pain during dressing changes.

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823 039 (07/17)

HydroTherapy
Efficacy. And Simplicity.

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